



Windows and Dialogs

Display	Key	Tool
Close a Tab in the Chat Window	Ctrl+W	
Close the Chat Window	Alt+F4	
Extension Window	E	
Help Documentation	F1	
Image Layout Dialog		
Key Image Composer	K	
Print Film/Show Print Film	Ctrl+P	
Report Viewer	V	
Search Tool	?	
Series Layout Dialog		
Show or Hide Thumbnail View	F11	
Switch to the Next Tab in the Chat Window	Ctrl+Tab	
Teaching Case Editor	F	
Thumbnails	T	
User Preferences	Ctrl+U	

Manipulating Images

Function	Key	Tool
Add Image to Print Film	Shift+Spacebar	
Adjust Image Using DSA	X	
Add Key Image	Spacebar	
Add Teaching Case Image	Ctrl+Spacebar	
Add Text/Arrow/Ellipse Annotation	A	
Auto Linked Stacking	L	
Complete Annotation/Measurement	Esc	
Cycle Auto Window Level Presets	Ctrl+F2	
Cycle DICOM Presets	Shift+F2	

Function	Key	Tool
Display CAD Marks (Mammography Viewer)	* (NumPad)	
Display Shutters	Ctrl+R	
Drag and Swap	D	
Export Images	Ctrl+Shift+E	
Flip Horizontal	Ctrl+H	
Flip Vertical	Ctrl+V	
Increase Series Quality	~	
Increase Series Quality to Lossless	Ctrl+~	
Invert Image Colors	I	
Link Viewports	Ctrl+click	
Lock/Unlock Viewports	Ctrl+L	
Magnifying Glass	G	
Manual Linked Stacking	L	
Open DICOM File	Ctrl+O	
Pan	P	
Quadrant Magnification (Mammography Viewer)	Q	
Quick Export	Ctrl+E	
Reset Image	Ctrl+ Backspace	
Rotate Left 90°	<	
Rotate Right 90°	>	
Stack	S	
Toggle Super Series Stacking		
Toggle Text Overlay	O	
View Header	H	
Window Level	W	
Zoom	Z	
Zoom 100% (Mammography Viewer)	Ctrl+O	






Image Navigation

Function	Key
Cine	Ctrl+I
Display Next Series	Page Down
Display Previous Series	Page Up
Page Forward to Next Series	Right Arrow
Page Backward to Previous Series	Left Arrow
Display First Image	Home
Display Last Image	End
Page Image Forward (multi-image layout)	Down Arrow
Page Image Backward (multi-image layout)	Up Arrow

Mammography Image Navigation

Function	Key
Page Forward to Next Step	Right Arrow
Page Backward to Previous Step	Left Arrow
Display First Step	Home
Display Last Step	End
Display Oldest Comparable Study	Down Arrow
Display Most Recent Comparable Study	Up Arrow
Cycle through next tomosynthesis and mammography images in a study	Ctrl+Down Arrow
Cycle through previous tomosynthesis and mammography images in a study	Ctrl+Up Arrow

3D and Measurement Tools

Function	Key	Tool
Show 3D Cursor	C	
Show Reference Lines	R	
Add Measurement	M	
Calibrate Measurements	Ctrl+M	
Delete Selected Measurement	Delete	

Series and Image Layout

Series Viewports (on current monitor)	Key
1	1
2	2
3	3
4	4
6	5
9	6
12	7
16	8
18	9

Number of Images (in selected viewport)	Key
1	Ctrl+1
2	Ctrl+2
3	Ctrl+3
4	Ctrl+4
6	Ctrl+5
9	Ctrl+6
12	Ctrl+7
16	Ctrl+8
18	Ctrl+9

Server Addresses

URL: <https://pacs.svcmi.com.au>